


Important information on the vacuum safety indicator!


Mode of operation of the Mobeli safety rocker: The vacuum is permanently monitored.

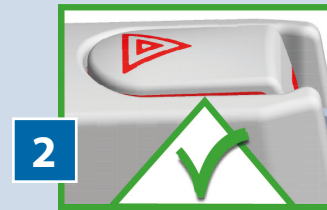

1 Full security for the specified maximum user weight. Fig. 1+2



3 Attention: The handle must be reattached immediately! The vacuum of the handle is reduced. The handle is no longer permissible for the maximum user weight. (Only for your information: A reduced holding force is still present.) Fig. 3+4

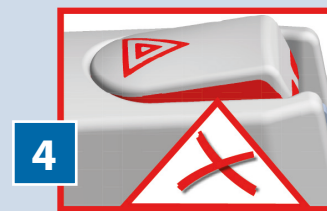

5 If a handle is attached to a non-porous dense surface for a long period, the rubber gets stretched and slightly curved (Fig. 5).

We therefore recommend that you remove the handle from the surface approx. every 14 days and leave it overnight on a flat surface with the rocker arms relaxed and reattach it the next morning.

When re-attaching it is important to press the rocker arm firmly to the surface.

Observe the instructions overleaf.

If the safety rocker indicator moves out faster than usual at the same attachment point, we recommend a thorough cleaning.

If the rubber discs are extremely overstretched or scratched - they should be replaced. Contact your dealer.

If the safety rocker is still raised after the rocker has been turned and also is still sticking out after the tab has been pressed, you should attach the handle again.

If the safety rocker is extended as far as shown in Figs. 3+4, the handle must not be used any further. Risk of accident!





Attention. Keep instructions!



- The mobile grab handles are intended to support the natural movement, e.g. as assistance for getting up. They are not designed to carry the full body weight. Please observe the maximum user weight specified in the instructions and on the handles!
- Before each use of the portable grab bar the vacuum safety indicator must be examined and handled in accordance with the directions in instructions.
- In order to avoid any damage or injuries due to incorrect use and to ensure optimum use of the product, **please read the following instructions and hints carefully before using the device.** Please retain the instruction manual and ensure that it is at the disposal of all other or subsequent users. If you have additional questions or problems, please contact your supplier.



1

Firmly press the mobile grab handles onto the wall.



2

While pressing the bar against the surface to be adhered to and also pressing on the rocker.



3

Turn over the 1st rocker and then the 2nd rocker.

Please see overleaf and read supplied instructions thoroughly!



ROTH Info Leaflet D14.812.23.03 20230307 EN

Subject to technical modification.

www.mobeli.de
Please contact us: info@mobeli.de

ROTH®

ROTH GmbH
D-72213 Altensteig
Telefon +49 (0) 7453 9381-0
Fax +49 (0) 7453 9381-23



Always keeping an eye on your safety.